

# VETERINARY: TREATMENT OF A TAIL WOUND (CAT)

t=3 days



18-8

Severe wound located approximately 2 cm distal of the caudal peduncle, ventral side, about 10 cm long. Injury caused muscles to peel, veins were still intact. Necrosis and granulative tissue were present. Wound was cleaned before treatment.

Goal of treatment: Create a closed wound environment to reinforce healing capability of the tissue.

Pictures are 3 days into treatment with MelMax 10x10 cm, covered with a regular gauze.

## Product used:

MelMax (MM050610 & MM081010)  
Sterile honey impregnated wound dressing.



t=9 days



24-8

Wound volume reduced significantly, wound surface further reduced, allowing further treatment with MelMax 5x6 cm.

Tail regained normal form (bending backwards) and function.

No further need for a collar, the wound dressing was tolerated well.

*MelMax applied to the wound (18-8)*

